

The holidays are approaching – what should I do?

Checklist

Product	Action	
Suction System	<ul style="list-style-type: none"> <li data-bbox="453 416 1034 577"> ■ Disinfect and clean the suction system with Orotol plus and MD 555 cleaner (observe the correct concentration and contact time) <li data-bbox="453 667 1200 745"> ■ Press the spittoon valve cleaning button and clean the yellow protective screen (replace if necessary) <li data-bbox="453 813 1295 853"> ■ Replace the disposable yellow filters on the hose holders <li data-bbox="453 1133 1023 1211"> ■ Clean the yellow filter at the suction system inlet (wet suction systems only) <li data-bbox="453 1283 1088 1361"> ■ Replace the amalgam collection container on the amalgam separator <li data-bbox="453 1570 1241 1693"> ■ Remove all suction handpieces from the hose holder to start the suction system. Run the suction system for at least 30 minutes to dry the pipework <li data-bbox="453 1749 1318 1870"> ■ Disconnect all suction handpieces from the suction hoses; clean and disinfect the handpieces in accordance with the manufacturer's instructions 	<div style="display: flex; flex-direction: column; align-items: center; gap: 20px;"> <input data-bbox="1398 416 1458 479" type="checkbox"/> <input data-bbox="1398 667 1458 730" type="checkbox"/> <input data-bbox="1398 813 1458 875" type="checkbox"/> <input data-bbox="1398 1133 1458 1196" type="checkbox"/> <input data-bbox="1398 1283 1458 1346" type="checkbox"/> <input data-bbox="1398 1570 1458 1632" type="checkbox"/> <input data-bbox="1398 1749 1458 1812" type="checkbox"/> </div>



The holidays are approaching – what should I do?

Checklist

Product:	Action	
Compressor	<ul style="list-style-type: none"> Wait until the compressor has come to a complete stop 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Open the condensate drain valve on the tank 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Wait until the start-up pressure is reached and the compressor restarts 	
	<ul style="list-style-type: none"> Open the condensate discharge valve on the membrane dryer until air and water are expelled. 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Once no more water is discharged, you may close the condensate discharge valve 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Switch off the compressor – via the electronic control or the main switch on the rear 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Wait until all hissing stops, then close the condensate discharge valve on the tank. 	<input type="checkbox"/>
	<ul style="list-style-type: none"> Unplug the power cord 	<input type="checkbox"/>

